

PLAQ™ Instructions for use

Preparation

Take a PlaQ tray and place it in on a hygienic clean surface. Stabilize the PlaQ tray with one hand and remove the seal foil with the other hand. Get disposed of the foil before proceeding. Place the PlaQ tray on a dental treatment tray. The product is now ready for use. In some cases moisture may appear on the inside of the seal foil. This is of no influence on the quality or functionality of the product.

Clinical procedure

Let your patient brush and interdental clean his or her teeth. Apply the dental plaque disclosing solution on all surfaces of the teeth with the special PlaQ applicator included. Let your patient rinse the mouth once with water. You can now show your patient the areas where plaque still is present and more thorough cleaning is needed. The stained dental plaque can easily be removed by polishing, toothbrushing and interdental cleaning of the teeth.

Explanation

PlaQ™ contains DentalColors two-tone dental plaque disclosing liquid. It effectively stains old dental plaque (> 24 hours) purple and new plaque in a pink/red color. Thereby making it visible to you and your patient. PlaQ shows the areas where more thorough toothbrushing and better interdental cleaning are needed. Your patient will thereby be able to achieve good oral hygiene. PlaQ also helps children learn how to brush more effectively.

Composition and use

Active ingredients of PlaQ: D&C Red 28 (1,5% w/w), purified water.
Only to be used by dental professionals. Do not swallow.

Precautions

- To prevent cross contamination PlaQ may only be used during one treatment session of one patient (single patient use only). PlaQ is a disposable product.
- If PlaQ liquid comes in contact with the eye, rinse immediately with water and contact a doctor.
- If PlaQ liquid comes in contact with skin or clothes, clean the contaminated surfaces with water and soap.

Storage

PlaQ is not be used after the expiration date. Do not store below 10°C or above 25°C. Avoid exposure to sunlight.

Garanty

- PlaQ™ and DENTALCOLORS® are trademarks of DENTALCOLORS BV
- DENTALCOLORS BV replaces all defective products. We guarantee the quality of our products in accordance with the existing standards.
- DENTALCOLORS BV products have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and / or written information has been given to the best of our knowledge and without obligation. Our information and / or advice do not relieve you from examining the materials delivered by us as to their suitability or the intended purposes of application. As the application of our products is beyond our control, the user is fully responsible for the application.
- DENTALCOLORS BV products are only to be used by qualified dental professionals.

DENTALCOLORS BV

Kanaalstraat 235

7547 AS Enschede (NL)

T: 085-7732132

E: info@dentalcolors.com