

# CARIEZ™ Instructions for use

---

## Preparation

Pour some drops of CarieZ liquid in a dappendish and place it on your treatment tray.

## Clinical procedure

First remove the softened and clearly discoloured dentin. Rinse the cavity with water and slightly dry it. Then wet the cavity with CarieZ detection liquid and allow it to act for 5 to 10 seconds (not longer). Rinse the liquid away with water spray. Infected, carious dentin will turn bright pink. Remove the dyed dentin areas completely, taking care to conserve the undyed dentin areas. Repeat the application of the CarieZ detection liquid as mentioned above until no further dentin colouration occurs. Plaque and residual caries in enamel will equally be dyed pink by CarieZ and should also be removed.

## Explination

By using the aforementioned technique the dental practioner can make sure that all infected, carious dentin will be removed and healthy dentin will be preserved. Healthy as well as uninfected, remineralisable dentin will not turn pink, enabling gentle and secure excavation. CarieZ prevents pulp irritation by providing a clear boundary between infected and healthy dentin.

## Composition and use

Active ingredients of CarieZ: coloring pigment, propane-1,2-diol.  
Only to be used by dental professionals. Do not swallow.

## Precautions

- If CarieZ liquid comes in contact with the eye, rinse immediatly with water and contact a doctor.
- If CarieZ liquid comes in contact with skin or clothes, clean the contaminated surfaces with water and soap.

## Storage

CarieZ is not be used after the expiration date. Do not store below 10°C or above 25°C. Avoid exposure to sunlight.

## Garanty

- CARIEZ™ and DENTALCOLORS® are trademarks of DENTALCOLORS BV
- DENTALCOLORS BV replaces all defective products. We guarantee the quality of our products in accordance with the existing standards.
- DENTALCOLORS BV products have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and / or written information has been given to the best of our knowledge and without obligation. Our information and / or advice do not relieve you from examining the materials delivered by us as to their suitability or the intended purposes of application. As the application of our products is beyond our control, the user is fully responsible for the application.
- DENTALCOLORS BV products are only to be used by qualified dental professionals.

## DENTALCOLORS BV

Kanaalstraat 235

7547 AS Enschede (NL)

T: 085-7732132

E: [info@dentalcolors.com](mailto:info@dentalcolors.com)